



User Manual

BR01

Wireless BathroomWatcher™



Product Description

Rondish wireless BathroomWatcher™ emergency call system delivers sound and light alert signal to caregiver when a resident pulls cord or presses the call button. When the resident is attended to, caregiver can then reset the alarm by pressing reset button.

It complies with government regulations devised to ensure barrier free entry and use of public facilities by the disabled.

What's in the box

Emergency Alarm ILB-03

Wireless Call Button WCP-03

Wireless Reset Button WRE-03

Caregiver Key CK-01

D-type Alkaline Battery x 4

User Manual BR01

1-Year Warranty Card

Items Needed

Screwdriver (+/- Style)

Installation Guide

Important:

1. Drilling and wall mounting techniques are required during installation, it is highly recommended to install this product by qualified technician.
2. Wireless connection between alarm and button is preset before shipment. To avoid disconnection, always keep wireless call button and reset button within 25 meters (82 feet) during installation.

A. Power Supply

Using D-type Alkaline Battery

1. Use a screwdriver to open battery case behind emergency call alarm
2. Insert 4 D-type alkaline batteries

Using AC Adapter

1. Connect 9V adapter to electric power socket
2. Plug power cord into 9V electric slot on emergency call alarm

** Using battery and AC adapter at the same time will not cause any damage to the product*

B. Test the Alarm

1. Press wireless call button or pull emergency cord to trigger alarm
2. Adjust alarm volume at the back of emergency alarm
3. Press wireless reset button to reset alarm

C. Install Emergency Call Alarm

1. Install emergency call alarm outside bathroom at a visible and open area on a wall, using punch holes at the back

D. Install Wireless call button and reset button

1. Install wireless call button and reset button inside bathroom using punch holes.

Note:

It is recommended to install wireless call button and reset button next to each other, and approximately 1 meter from the floor. Emergency pull cord shall be hanged 30 centimeters above the floor.

Setup Wireless Connection

Note:

- Wireless connection is only required to setup when adding a new button or device connection is lost
- Emergency alarm should be powered before setting up wireless connection

1. Enter Pairing Mode

- Press and hold "Program(Reset)" button at the back of emergency alarm until low battery light turns on.

2. Activating Alarm Call Button and Reset Button

- Place and hold magnetic caregiver key on small triangle mark at top right corner of alarm call button until red light turns on.
- Wireless connection is setup after several beep sound followed by a steady light on emergency alarm.
- Repeat the above 2 steps for reset button

3. Exit Pairing Mode

- Press and hold "Program(Reset)" button at the back of emergency alarm. Pairing mode will be turned off after two short beeps.
- Alarm call button can be tested after "Low Battery" light turns off.